**WARNING:**

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 300 POUNDS. Consult healthcare professional for advice and instructions specific to your needs before use. Plastic inner rim of seat must fit inside of porcelain toilet bowl rim. If seat can rock from side to side, do not use. Do not tip seat. Sit down on and leave seat from front, not from side. When seated, do not lean to either side of seat without proper balancing. Do not jump or stand on seat. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

**LIMITED LIFETIME WARRANTY**

If this product fails due to a defect in materials or workmanship at any time during the life of the product, CSI will replace it free of charge, postage-paid. Simply contact CSI Customer Service at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OF RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

The Home Care Locking Elevated Toilet Seat meets or exceeds the applicable requirements of the following standards/guidelines:

The Americans with Disabilities Act (ADA)

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The Americans with Disabilities Act (ADA)

Creative Specialties is a division of Moen Incorporated. © 2009  
Creative Specialties is a division of Moen Incorporated. © 2009  
Creative Specialties est une division de Moen incorporée. © 2009

INNOVATIVE DESIGNS  
*for independent living™*

**LOCKING ELEVATED TOILET SEAT WITH  
SUPPORT HANDLES INSTRUCTION SHEET**

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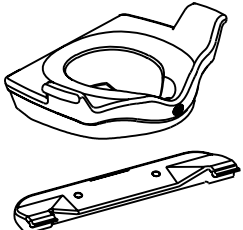
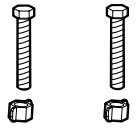
# Locking Elevated Toilet Seat with Support Handles Instruction Sheet

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**Warning ! – FOLLOW THESE INSTALLATION INSTRUCTIONS CAREFULLY. PROPER INSTALLATION IS THE INSTALLERS RESPONSIBILITY. FAILURE TO FOLLOW CORRECT INSTALLATION PROCEDURES CAN CAUSE THE SEAT TO BECOME LOOSE, WHICH CAN RESULT IN SERIOUS INJURY.**

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### Specifications:

Fits on most commode configurations – either round or elongated bowl. Raises seat 5 inches. Toilet seat must be clean and dry before installation. Plastic rim of seat must fit inside of toilet rim.

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61 cm larg. x 41 cm prof. x 13 cm haut. (20.5 x 16 x 5 po)

### Installation Instructions:

Tools Required: Standard size screw-driver to attach the bracket.

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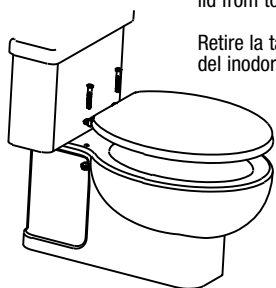
1

Remove standard toilet seat and lid from toilet bowl

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2

Place bracket on toilet bowl and align set of holes

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3

Re-attach toilet seat lid with mounting bracket using supplied screws by lining up the holes on the toilet bowl.

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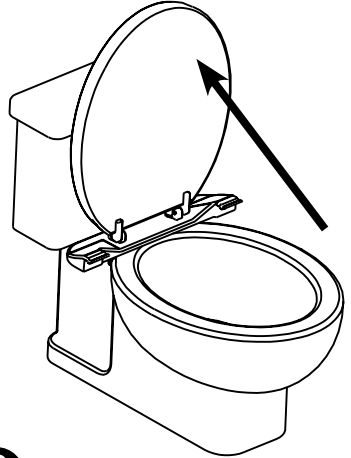
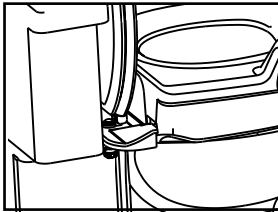
5

Attach the elevated toilet seat to the bracket on toilet bowl.

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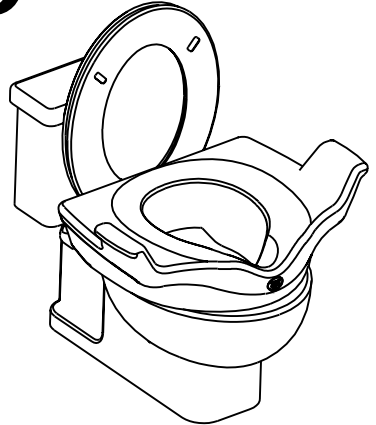
Push the seat towards the bracket to lock

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To remove, reverse process, lift front of seat and pull

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